



# Use and Care Instructions

## Diamond Blade Cleaning System

---



Mastel's Diamond Blade Cleaning System is sold by the case. Each case contains two boxes of 12 cleaning trays. We recommend using one cleaning tray per operating day, per room.

Cleaning tray use is outlined on page 2 & inside lid of each cleaning system tray.




Mastel also carries a reusable holding tray/cradle for the blade cleaning system trays. The holding tray provides extra stability to the cleaning system during use and storage.

## Daily instructions:

1. Select a new tray from the dispenser carton.
2. Insert tray in holding tray if available. Open lid and Peel foil seal from cleaning system. Close lid(s) to prevent solution evaporation.
  - » Keeping the plastic lid(s) shut between uses will extend tray life.
  - » Starting in a selected corner and working in ½ inch sections can help identify used areas of the tray and extend its life.
3. If the pad substrate becomes dysfunctional dispose of the tray and open a new cleaning tray.
  - » We recommend using the Diamond Blade Cleaning System (DBCS) to clean blades immediately after use.
4. At the day's end dispose of any used cleaning trays.



1. Open holding tray and/or tray lid.
2. Fully extend the blade and lock its position.
2. Carefully push blade and footplate (when applicable) into the detergent (blue), gently moving in the direction one would cut (i.e., side to side) 5-10 swipes.
  - » Avoid lateral motion
3. Then move to each of the rinses (white) using the same motion to remove the cleaning solution.
  - » A quick inspection under magnification can reveal any material missed. Repeat cleaning if necessary.
4. Once you are confident the blade and footplate are clean, retract the blade fully.
5. Return the scalpel to its dedicated sterilization tray and proceed to the sterilization process.
  - » Always store scalpel in the case with blade in retracted, locked position. Make sure blade end is pointing towards largest end of case.

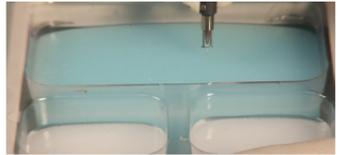

**Diamond blades are very fragile!**

Make sure the blade edge(s) do not come in contact with other instruments or the bottom of the cleaning tray during this process.

## Product Numbers:

---

Balde Cleaning System Case      # SUOK C  
(24 per Case)

Holding Tray/Cradle                      # SUOK T

---



and its team of craftsmen, engineers and service representatives strive to provide you with the very best in ophthalmic surgical instruments. Our products are delivered to you with pride.

If, for any reason, you need further assistance, please call one of our service representatives. They will provide the help you seek.

Thank You!

## Certifications & Licenses

Mastel is CE listed and  
FDA registered.

### Certifications:

- ISO 13485:2016 (MDSAP)
- EN ISO 13485:2016

### Licenses:

- Health Canada



2843 Samco Rd. Ste. A  
Rapid City, SD 57702

[www.mastel.com](http://www.mastel.com)

### Phones:

Main                      605-341-4595  
Toll Free                800-657-8057  
Fax                        605-343-3631  
Email                    [sales@mastel.com](mailto:sales@mastel.com)